



Erycard™ 2.0
Blood Grouping Card for ABO/Rho(D) Forward Grouping with Autocontrol

Intended Use: _____

Erycard™ 2.0 can be used as

- Bed side ABO & Rh testing of Patient and Donor's blood unit before transfusion
- Donor's blood group screening in outdoor donation camps
- Outdoor blood grouping camps
- Routine laboratory testing
- Preconfirmation test, especially at nursing homes
- For releasing compatible blood for emergency transfusions in
 - Defence Camps
 - Ambulance
 - Casualty services

Erycard™ 2.0 Kit components:

Device	Pre-dried reagent at the appropriate sample pad Anti-A (IgM) monoclonal antibodies in sample well A Anti-B (IgM) monoclonal antibodies in sample well B Anti-D (IgM) monoclonal antibodies in sample well D Autocontrol without any antibodies (Ctrl)
Buffer	Ready to use Reagent Buffer (Proprietary Buffer) in a dropper bottle
Other Accessories	Desiccant pouch, Sample Collection Loops and Package Insert

Performance Characteristics:

Sensitivity & Specificity: 100% correlation with classical tube technique	Standardization: Follows FDA guidelines
---	---

FEATURES & BENEFITS
Blood Grouping in less than 5 minutes
Easy 3 step procedure
Simple interpretation of results
Auto control for test validation
Room temperature storage

Storage / Stability	Temperature	Duration
Unopened kit	2-30°C	24 months
Opened device	R.T.	Use immediately

Available Pack Sizes
24 Cards